

NOCOSPRAY USER MANUAL



CAREFULLY READ THE ENTIRE USER'S MANUAL THAT COMES WITH THE NOCOSPRAY ROBOT!



1 Use gloves and eye protection when handling any chemicals used for the robot.




2 Check that the bottle contains sufficient amount of chemistry. Mark the level *before* the procedure and check *after*.



3 Secure the spray nozzle onto the bottle.
Keep the cap!



4 Attach the spray nozzle with the bottle in place on the Nocospray robot. Ensure the nozzle is firmly attached.

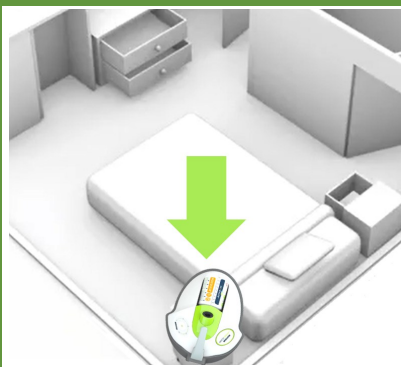

5 Place the Nocospray robot on a stable surface. Choose a corner for optimal effect.



Nocolyse OneShot: room volume (m³) **x3**
Nocolyse Neutral: room volume (m³) **x1**

6 Set the correct room volume. The setting depends on the size of the room and the type of chemistry used.



7 Press start in the center of the dial. The robot will beep for 15 seconds and start automatically.

Leave the room !



Nocolyse OneShot: **2 hours** contact time
Nocolyse Neutral: **30 min.** contact time

8 The room must remain closed/locked during diffusion and throughout the contact time. Hang a poster on the door with a warning about "No entry!"



9 Detach the bottle and replace the cap. Rinse the spray nozzle and tube under water. Click the nozzle back into place in the robot.