NOCO SPRAY 2

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NO

SAMMENDRAG

NO

1. INNLEDNING 5

1.1 Generelt.....5

1.2 Klassifisering / Forskrift5

1.3 Garanti.....5

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3. BESKRIVELSE 8

3.1 Beskrivelse av NOCOSPRAY 28

3.2 Tilstopping8

4. BRUK 9

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4.2 Plassering av flasken9

4.3 Drift i standardmodus10

4.3.1 Bruk uten fjernkontroll.....10

4.3.2 Bruk med fjernkontroll11

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6.4 EMC samsvarstabell

NO

EMC Compliance according to IEC 60601-1-2 (2007)			
The NOCOSPRAY 2 is intended for use in the electromagnetic environment specified below.			
The customer or the user of the NOCOSPRAY 2 should assure that it is used in such an environment.			
Emission test	Standard	Compliance	Electromagnetic environment
Emissions RF	CISPR 11	Group 1	The NOCOSPRAY 2 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
Emissions RF	CISPR 11	Class B	The NOCOSPRAY 2 is suitable for use in all establishments, including domestic establishments and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.
Harmonics emissions	IEC 61000-3-2	N/A	
Voltage/Flicker test	IEC 61000-3-3	N/A	
The NOCOSPRAY 2 is intended for use in the electromagnetic environment specified below. The customer or the user of the NOCOSPRAY 2 should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment
Electrostatic discharges IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical Fast Transients IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % U_T (>95 % dip in U_T) for 0,5 cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5 s	<5 % U_T (>95 % dip in U_T) for 0,5 cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the system requires continued operation during power mains interruptions, it is recommended that the system be powered from an uninterruptible power supply or a battery. U_T is the a.c. mains voltage prior to application of the test level.
Magnetic field at power frequency IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Conducted RF IEC 61000-4-6	3 Vrms 150kHz-80MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the NOCOSPRAY 2, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance : $d = 1,67 \cdot \sqrt{P}$
Radiated RF IEC 61000-4-3	3V/m 80MHz-2.5GHz	3V/m	$d = 1,67 \cdot \sqrt{P} \text{ 80MHz-800MHz}$ $d = 2,33 \cdot \sqrt{P} \text{ 800MHz-2,5GHz}$ where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey (a) should be less than the compliance level in each frequency range (b) Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

(a) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the NOCOSPRAY 2 is used exceeds the applicable RF compliance level above, the NOCOSPRAY should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the NOCOSPRAY 2.

(b) Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.

Recommended separation distances between portable and mobile RF communications equipment and the NOCOSPRAY 2.

The NOCOSPRAY 2 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the NOCOSPRAY 2 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the NOCOSPRAY 2 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150kHz to 80MHz	80MHz to 800MHz	800MHz to 2.5GHz
0.01	0.117	0.117	0.233
0.1	0.369	0.369	0.737
1	1.167	1.167	2.330
10	3.690	3.690	7.368
100	11.67	11.67	23.300

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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